


microMend[®] PRO

Skin Closure Device


Now Available for Use in Your Facility

WOUND MUST BE COMPLETELY DRY BEFORE APPLICATION


Simple to Apply: **PRESS - PULL - SLIDE - PULL - PRESS**

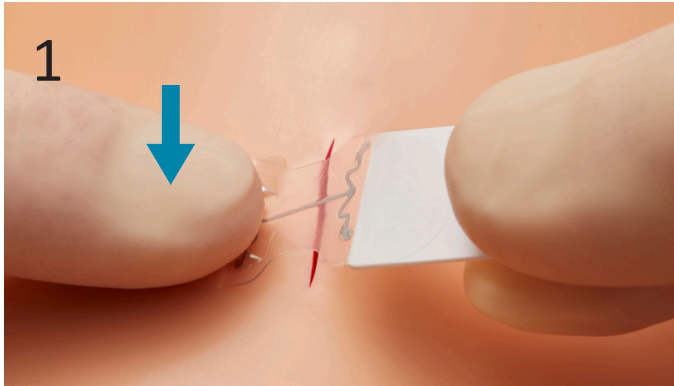


PRESS on Distal Side (away from White Tab), including over Microstaples.

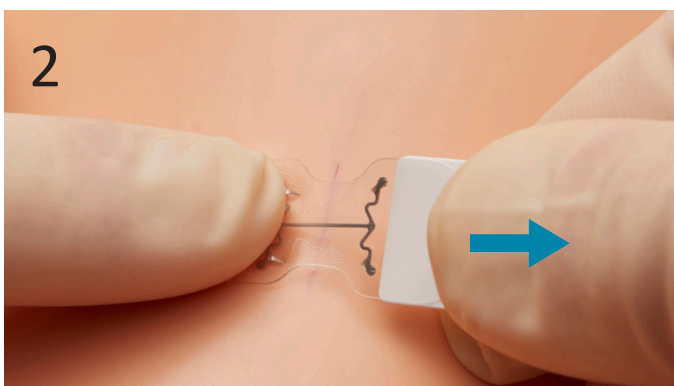


While pressing down, **PULL** Tab to bring wound edges together.

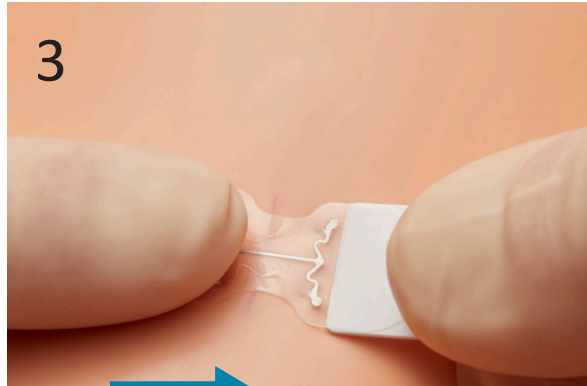


SLIDE FINGER ACROSS

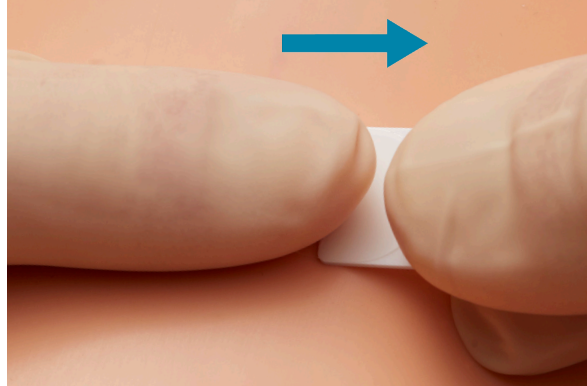

While keeping finger in contact with device, **SLIDE** finger across device.

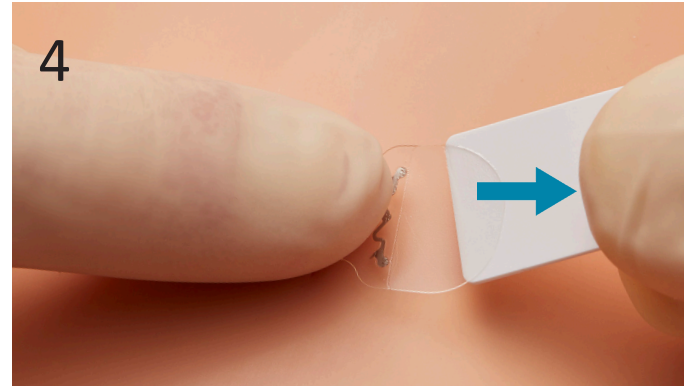

Pressing only over Microstaples, **PULL** Tab along skin surface to remove it.

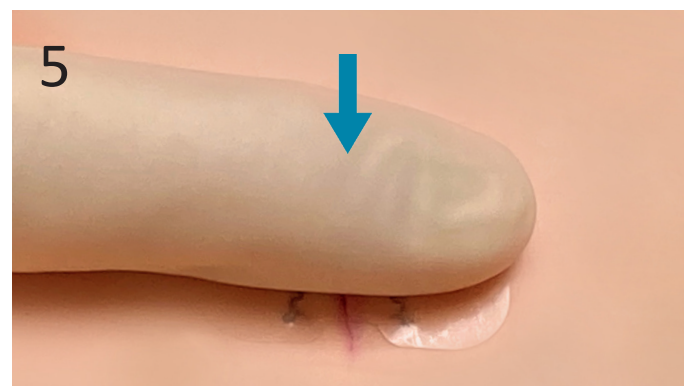


PRESS over both sides of entire device to secure microMend PRO.

Cover wound area (including microMend PRO) with a non-stick dressing.

Quick and Easy as a Bandage with Holding Strength of Sutures

- Painless and easy to apply
- Much faster than sutures
- Excellent cosmetic results
- Patients can remove at home
- Significant cost savings
- Preferred by physicians and patients
- Same application method for Small, Medium and Large devices

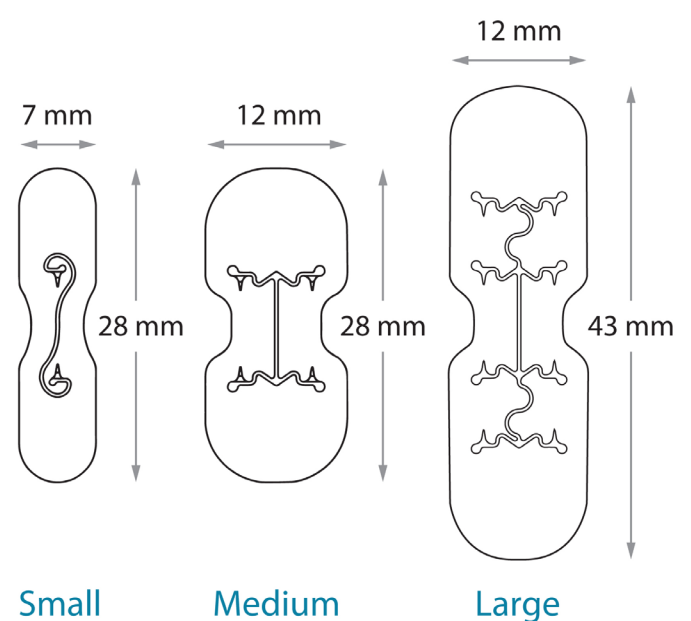

Additional devices available for longer wounds and umbilical port sites



SCAN HERE

View Video How to Apply

Complete Instructions for Use (IFU)
and Warnings Included in Product Package

Made in USA